

Crutch Holder

B6213

DIESTCO Crutch Holder mounts to the back of your scooter or powerchair with a 2" hook and loop strap. Keeps your crutches close, but out of your way. Made from high quality, water-resistant polyester with 300-yard visible reflective safety tape.

Fits seatbacks 7" - 24" tall.

Mounting Instructions:

- 1) Remove the strap from the metal loop.
- 2) Pass the strap through the opening between the seat and the seatback.
Be sure to pull the strap through completely and that it is not twisted.
- 3) Re-insert the strap through the metal loop and pull through.
- 4) Pull strap down tightly and fasten the loop end to the hook portion of the strap.
- 5) Excess strap can be cut once you have your Crutch Holder mounted.

If you have any issues mounting the holder, please call us at (800)795-2392 for assistance.



DIESTCO Mfg. Corp.
P.O. Box 6504
Chico, CA 95927
(800) 795-2392
www.diestco.com

We strongly encourage you to call us at **(800)795-2392** if you have any problems, questions, or comments. We are always developing new products for wheelchairs, scooters, crutch, and walker users...so let us know if there's something special you need.